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### INSTRUCTIONS FOR USE:

ENG

**Indications For Use:** It is a disposable spring loaded biopsy device used for histological core samples from soft tissue and organs. GEOTEK semiautomatic biopsy needle has two advancement options: 1cm and 2cm.

**Contraindications:** This needle should be used by a physician trained in interventional techniques and familiar with the possible side effects, typical findings, limitations, indications and contraindications of biopsy procedures. Physician judgment is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications.

**Possible Side Effects:** Pain, fever, bleeding, hematuria and infection.

**Caution:** Ethylen oxide sterilized. Non-pyrogenic. Check the package. if package damaged or opened do not use the products. Check the expiry date and the gauge. Store in dry place, between 5 – 30 °C temperature, protect from sun light. Possible allergic reactions should be considered. It must be used only in hospitals by physicians who familiar with the device and procedures. For single Patient Use Only. Do not attempt to clean or resterilize this product. Do not use the device under MRI. After use, this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

Please double check for device function and possible damage for correct implantation and do not use if package is opened or damaged.

### BIOPSY PROCEDURE:

1. Identify the biopsy site preferably under US or CT.
2. Using the sterile technique, prepare the skin with antiseptic and drape.
3. If necessary, infiltrate the selected area with local anaesthetic.
4. Open the needle pouch and remove protective sheath from the needle.
5. Charge Biopsy needle by pulling back hard on color coded plunger in a horizontal direction until a clicking sound is heard. **Fig.1**

\**This needle has two advancement: 1cm and 2cm. The first clicking sound indicates setting of the 1cm option , and the second 2 cm option. Fig.2 and Fig.3*

6. Insert the biopsy needle and advance into the targeted area while the plunger is in backward position.

**Warning:** Avoid pushing the plunger during insertion of the needle

7. Move the plunger with thumb to advance the stylet and expose the specimen notch.

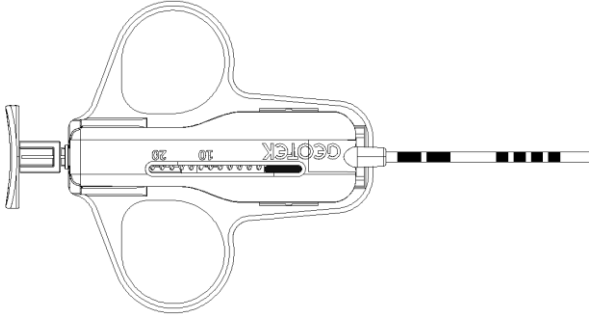
\**After insertion of the stylet into position it is possible to collect specimen more successfully by moving the plunger back and forth a few mm.*

8. Press plunger past stop with thumb. This will automatically trigger the cannula to close sample notch and trap the specimen.**Fig.4**

9. Draw needle out. Pull back on color coded plunger until it is locked. Push plunger firmly to expose the specimen notch. Remove the biopsy specimen from notch in stylet.

**Note:** These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

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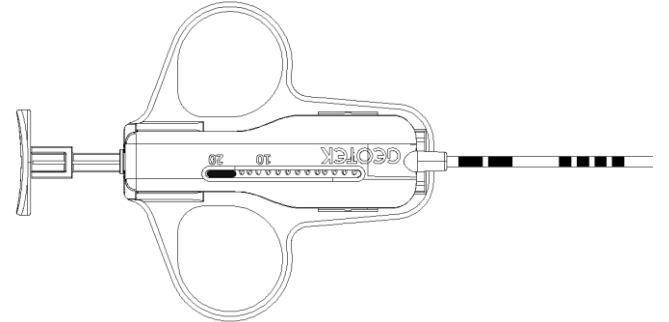
Fig.3: Adjustable sample notch:20mm  
*Parça alma boyu 20mm ayarlama*

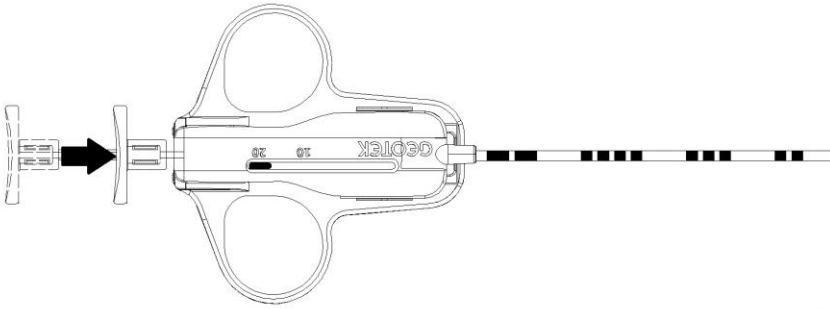
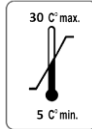
Fig.4: Firing the needle  
*Ateşleme*

Single Use    Storage limits    Expiry Date    Ethylen oxide sterilized

1984 : CE mark and NB denification number